

**DSHS Laboratory - Serology Specimen Acceptance Criteria (Effective 10/1/2016)**

Testing Requirements		Shipping Requirements			
Analyte	Test	Shipping Specimen Type	Shipping Requirement	Received Within	Temp
HIV Ag-Ab Multiplex	Immunoassay	Serum	Cold	5 days	2-8°C
			frozen / dry ice	> 5 days	-20°C or lower
HIV-1 / HIV-2 Supplemental	Geenius	Serum	Cold	5 days	2-8°C
			frozen / dry ice	> 5 days	-20°C or lower
Syphilis screen IgG	BioPlex_IgG	Serum	Cold	5 days	2-8°C
			frozen/dry ice	> 5 days	-20°C or lower
Syphilis	RPR Card test	Serum	Cold	5 days	2-8°C
			frozen/dry ice	> 5 days	-20°C or lower
Syphilis-confirmation	TP-PA	Serum	Cold	5 days	2-8°C
			frozen/dry ice	> 5 days	-20°C or lower
CT-GC	Amplified RNA Probe	Swab	Cold or ambient, up to 30 <sup>o</sup> C	30 days	2-30°C
		Urine	Cold or ambient, up to 30 <sup>o</sup> C	30 days	2-30°C
Hepatitis C	EIA - serum	Serum	Cold	5 days	2-8°C
		Serum	frozen / dry ice	> 5 days	-20°C or lower
Hepatitis A	Hep A Total Ab EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Hepatitis A	Hep A IgM EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Hepatitis B	Hep B Core Total Ab EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Hepatitis B	Hep B Core IgM EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Hepatitis B	Hep B Surf Ag EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Hepatitis B	Hep B Surf Ab EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Rubella screen	BioPlex_IgG	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Rubella	IgM EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Measles	IgG EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Measles	IgM EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Mumps	IgG EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Hantavirus	IgG/ IgM EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
West Nile / SLE	IgM - EIA	Serum/ CSF	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Dengue	IgM - EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Zika	IgM - EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Chikungunya	IgM - EIA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
QuantiFERON- TB	EIA	Serum	Cold, incubated and centrifuged	28 days	2-8°C
			frozen / dry ice, plasma only	> 28 days	-20°C or lower
Schistosoma or Strongyloides	IgG EIA	Serum	frozen / dry ice	As needed	-20°C or lower
Ehrlichia	IgG IFA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Rickettsial panel (RMSF, typhus)	IgG IFA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Q Fever	IgG IFA	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Brucellosis	MAT	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Tularemia	MAT	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Plague (Human)	PHI	Serum	Cold	2 days	2-8°C
			frozen / dry ice	> 2 days	-20°C or lower
Plague (Zoonosis)	PHI	Nobuto strips (Dried blood)	ambient temperature frozen/dry ice	3 months > 3 months	ambient temp -20°C or lower